# Premium Zahnschleifgerät # 22991

Meule à dents Premium  
Tooth Grinder Premium  
Molatrice per denti Premium  
Limador de dientes Premium

- DE Bedienungsanleitung
- FR Mode d'emploi
- EN Operating instructions
- IT Istruzioni per l'uso
- ES Instrucciones de uso

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---

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## Safety instructions

---

Read the following instructions carefully before using the device.

### CAUTION!

Danger of fire, short-circuit and electric shock!

- Do not use the connecting cable to disconnect the mains plug or to carry the device.
- Do not allow the connecting cable to come into contact with heat, oil or sharp edges.
- Store the device in a dry place and keep it away from liquids and moisture.

Danger of physical injury and material damage when used incorrectly!

- Ensure that the device is only used by people who have the relevant professional skills.
- Observe the maintenance instructions.
- Only operate the device with the appropriate mains voltage.
- Keep out of the reach of children.
- National and international safety, health and occupational safety regulations must be observed and complied with.

## Correct use

---

The device is used to file down piglets' teeth. Any other use is classified as incorrect. In the event of incorrect use or modifications to the device, the manufacturer's warranty and liabilities shall be deemed invalid.

## Operation

---

### Notes

- Ensure that the filing body is clean - see „Cleaning and care“. This is the only way of ensuring effective filing.
- Do not press the filing body hard against the teeth. This will interfere with the filing effect and put unnecessary strain on the filing body.
- File the tooth with only light pressure on the tip so that the sharp edges are removed.

Proceed as follows:

1. Switch the device on using the I/O button.
2. Set the speed. To do this, turn the selector knob to the appropriate setting (e.g. setting 8 corresponds to 8,000 rpm).
3. File the tooth. To do this, position the filing stone through the slit in the aluminium safety cap on the tooth.
4. Switch the device off again with the I/O button.

## Cleaning and care

---

Clean the filing body after each use. Clean the filing body and aluminium safety cap with a hard brush and water or vinegar. To thoroughly clean both parts, a conventional cold cleaner can also be used. After cleaning, grease the clamping sleeve and support for the cap nut (external thread).

## Maintenance

Check all components for damage before every use. Damaged parts should only be repaired by a qualified professional or the manufacturer. If repairs are required, only use original spare parts.

### Changing the filing bodies

Only use the original filing body (Art. No. 22914) in combination with the pre-fitted clamping sleeve (diam. 3.0 mm) as a replacement filing body.

Preceed as follows:

1. Unfasten the screw on the aluminium safety cap using a cross-head screwdriver and remove the aluminium safety cap.
  2. Open the clamping sleeve: To do this, press the locking button and simultaneously turn the cap nut using the enclosed key counter-clockwise.
  3. Remove the worn filing body and insert the new one.
  4. Close the clamping sleeve: To do this, press the locking button and simultaneously turn the cap nut using the enclosed key clockwise.
  5. Replace the aluminium safety cap and fix it in position with the screw.
- Caution!** The filing stone must not extend beyond the aluminium safety cap. If necessary, push the filing body further into the clamping sleeve.

## Why grind teeth?

The teeth of sucking pigs are sharp and pointed and can easily damage the sow's udder and teats. To protect the sow's udder and also to ensure calm and harmonious suckling it used to be customary to cut the teeth of sucking pigs. This also prevented bite injuries to the rest of the litter. Tooth cutting is a painful procedure for a sucking pig. When they are cut with a pair of tooth-nippers, even if the tooth is cut correctly, vertical cracks can easily occur in the dental enamel and the pulp of the tooth (nerves, blood vessels, connective tissue) becomes exposed. The cracks constitute a very high risk of future infection for the pigs. In many cases these infections are not discovered until slaughter.

## Technical data

|                  |                                     |
|------------------|-------------------------------------|
| Ref. No.         | 22991                               |
| Voltage          | 230 V, 50/60 Hz                     |
| Speed in rpm     | 5,000 – 20,000, variably adjustable |
| Consumption      | max. 100 watts                      |
| Protection class | II                                  |
| Weight           | approx. 500 g                       |

## Spare parts

|       |                      |
|-------|----------------------|
| 22992 | Aluminium safety cap |
| 22914 | Filing body          |

### Declaration of conformity

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations of Directive 2004/108/EC and 2006/95/EC. The CE mark confirms compliance with the Directives of the European Union. The declaration of conformity can be viewed by visiting <http://www.kerbl.com>.

### Electrical waste

It is up to the operator to dispose of the device responsibly after its useful life in accordance with local regulations. Consult the relevant regulations for your country. The device must not be disposed of in household waste. Under the terms of the EU Directive on the disposal of old electrical and electronic equipment, the device can be taken to local authority-run refuse sites or recycling plants free of charge, or returned to dealers who offer a returns service. Correct disposal protects the environment and prevents any possible harmful impacts on humans and the environment.

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---

## Warranty card

According to our General Conditions of Shipment for Products or Services we concede on the above appliance

## 1 year guarantee

Defect liability starts from the date of purchase.

This guarantee covers non of the following:

- 1) Damages caused by non-observance of the operation instructions.
- 2) Damages caused by force majeure as well as by transport.
- 3) Damaged rubber parts as e.g. power cords etc. are excluded from warranty.

**Repairs during the warranty period will only be carried out by the manufacturer!**

For this purpose the appliance has to be packaged properly and forwarded postage free at customer's risk together with the original invoice and the warranty card to the dealer (see dealer stamp). Please give detailed information on the kind of defect.

**Costs resulting from unjustified customer complaints are at the expenses of the customer.**

\_\_\_\_\_

Date

\_\_\_\_\_

Device-Number

Your dealer:

84428 Buchbach · Felizenzell 9 · Tel +49(0)8086/933-100 · Fax +49(0)8086/933-500

---

**Albert Kerbl GmbH**  
Felizenzell 9  
84428 Buchbach, Germany  
Tel. +49 8086 933 - 100  
Fax +49 8086 933 - 500  
info@kerbl.de  
www.kerbl.de

**Kerbl Austria Handels GmbH**  
Wirtschaftspark 1  
9130 Poggersdorf  
Tel. +43 4224 81555  
Fax. +43 4224 81555-629  
order@kerbl-austria.at  
www.kerbl-austria.at

**Kerbl France Sarl**  
3 rue Henri Rouby, B.P 46 Soultz  
68501 Guebwiller Cedex, France  
Tel. : +33 3 89 62 15 00  
Fax : +33 3 89 83 04 66  
info@kerbl-france.com  
www.kerbl-france.com